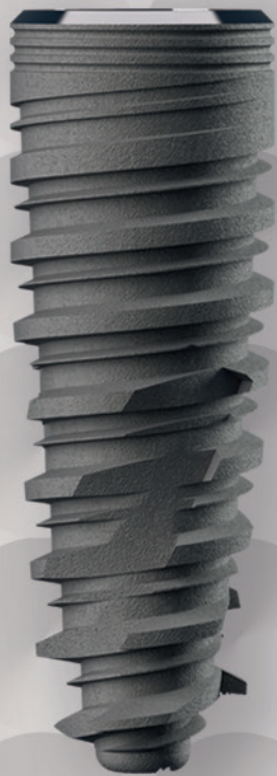
ed. 2018_1

UNIVERSE
Tapered drill
Protocollo chirurgico
Surgical Protocol

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Instruction for users

The UNIVERSE TD surgical protocol was developed to provide surgeons with the most appropriate tools for bone compliance, and is also simple and practical.

The preparation of the implant site for the Universe implant is completed in three simple steps, after which the implant can be inserted easily.

However, it is the duty of the surgeon to choose the most appropriate surgical protocol based on his or her experience following a thorough assessment of the individual patient's clinical situation.

The preparation of the implant site is performed by tapered drills that optimise the bone available to place the implant, without waste.

These drills are sized at the core of each single diameter and height of the implant to facilitate the drilling protocol reducing it to three simple steps.

The particular tip shape guides the progressive advancement, respecting the bone and preparing a customized site.

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Surgical Kit

The UNIVERSE TD surgical box is made entirely of plastic materials suitable for steam sterilisation.

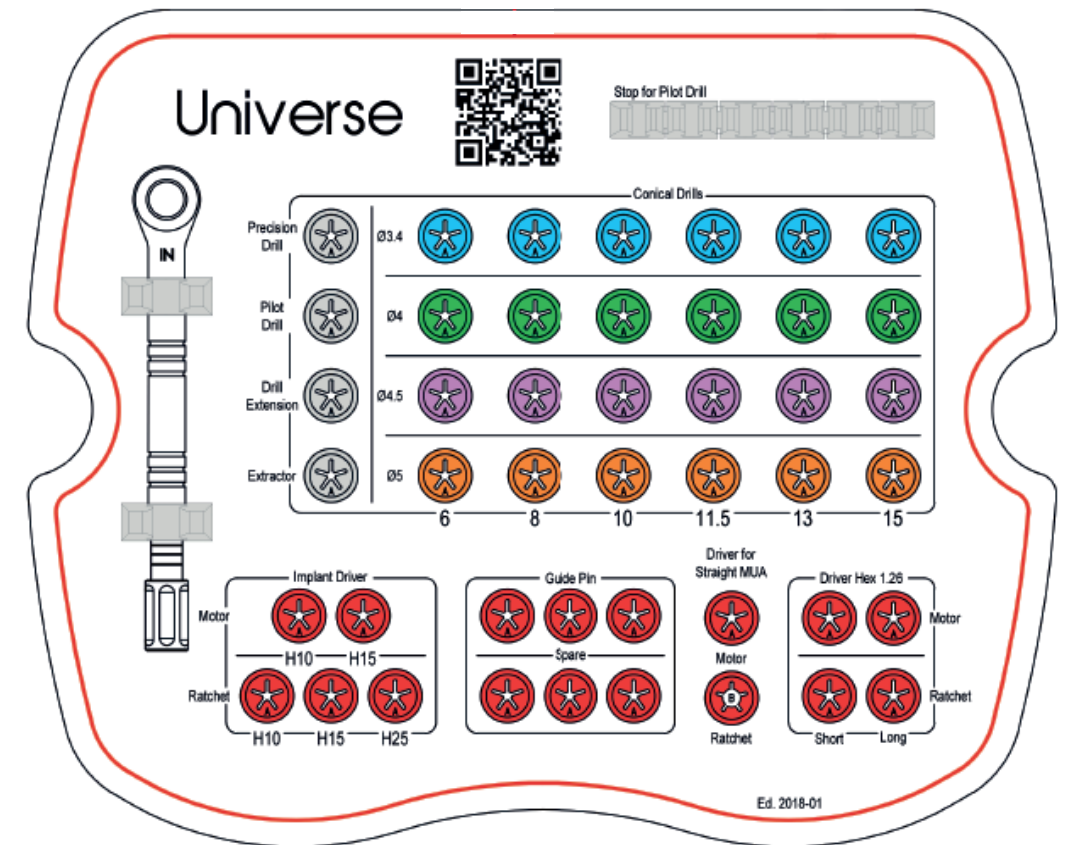
The instrument positions are clearly labelled in order to facilitate identification during the surgical operation.

The silicon supports secure the instruments firmly during transportation and sterilisation.

All IML surgical instruments are manufactured in Surgical Steel of the highest quality that offers the best performance in terms of wear resistance and torque.

Warning: drills prepare the site 0.3 mm more than the height of the implant.

Pilot drill is marked with indicators referring to implant height and drill stops.

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